





**Danny Ties**™  
Tracheostomy Ties

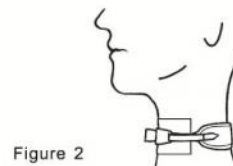

## Instruction for Use

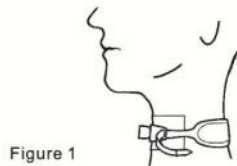

1. Insert the pointed hook end of the tie through the eyelet of the tracheostomy tube and attach it to the brushed side of the padded neck collar or to the strap (figure 1).
2. Wrap the padded neck collar around the patient's neck and then insert the other pointed hook end of the tie through the other eyelet of the tracheostomy tube. Pull the tie snug and attach it to the brushed side of the padded neck collar or the strap (figure 2).
3. Be sure to readjust the tie so that one finger can fit between the tie and the neck and make sure that the hook ends of the tie are secured to the brushed side of the padded neck collar.

**Caution:** When changing the tie on a patient without changing the tracheostomy tube, be sure to hold the tube in place while changing the tie to insure that decannulation does not occur.

For Single Patient Use. Danny Ties may be washed if soiled but please discard if the hook end of the tie does not have good adhesion to the brushed side of the tie.

**IMPORTANT NOTICE:** Please read the instructions for use carefully. The instructions should be read by all of those who apply this product. Always carefully monitor patients to ensure they are not at risk for disrupting lifesaving treatment. Patients must have constant supervision by trained personnel, electronic monitoring and/or the use of a restraint as well as proper medical supervision.